Drägerwerk AG & Co. KGaA, 23542 Lübeck

To our customers and users of the Fabius with
Auxiliary Common Gas Outlet (ACGO)

Add. device label – ext. fresh-gas outlet / aux. common gas outlet

September 2018

Dear Madam/Sir,

We would like to follow up on the Safety Notice issued by Dräger in October 2017, which addressed that the external fresh-gas outlet / auxiliary common gas outlet on anesthesia workstations had – in some cases – not been used in accordance to the instructions for use. Therefore, Dräger illustrated the functional principle of rebreathing systems and again highlighted guidelines for the safe use of the external fresh-gas outlet / auxiliary common gas outlet of Fabius anesthesia machines which are already given in the instructions for use.

Moreover, we already indicated our intention to create a separate device label as an additional reminder to check the setting of the fresh-gas path when starting a mechanical ventilation or ManSpont, (i.e. when using the COSY rebreathing system) in our initial writing.

You will find this new label and a supplement to the Instruction for Use enclosed to this letter. The label consists of a graphical and a textual part/description and it is designed to fit to the ventilator control panel next to the keys for selecting the ventilation modes. There is one label for each variant of the external fresh-gas outlet:

Drägerwerk AG & Co. KGaA
Moisinger Allee 53-55
23558 Lübeck, Germany
Postal address:
23542 Lübeck, Germany
Tel +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
VAT no. DE135082211

Bank details:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift-Code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift-Code: NOLADE21SPL

Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7903 HL
General partner: Drägerwerk Verwaltungs AG
Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7395 HL

Chairman of the Supervisory Board
for Drägerwerk AG & Co. KGaA
and Drägerwerk Verwaltungs AG:
Stefan Lauer
Executive Board:
Stefan Dräger (chairman)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske
Anton Schrofner

	Variant External fresh-gas outlet	External fresh-gas outlet with an additional switch
<i>Description in the IfU</i>	When an automatic ventilation mode (Volume Control, Pressure Control, Pressure Support, SIMV/PS) or the ManSpont ventilation mode is used,connect the short fresh-gas hose to the external fresh-gas outlet.	...set the switch on the external fresh-gas outlet to the COSY position.
<i>Label</i>		

Please select the label that fits your device configuration and the correct language and stick both onto the ventilator control panel as per the instructions on the sticker board.

In case of questions, please do not hesitate to contact your local Dräger contact.

Kind regards,

Alexander Herden

Product Management Life Support Systems
Anesthesiology

Attachment

- Labels for both variants of the ext. fresh-gas outlet
- Supplement to the Instructions for Use

Fabius family

de	Ergänzung, Seite 2	tr	Ek, sayfa 70
en	Supplement, page 6	el	Παράρτημα, σελίδα 74
fr	Complément, page 10	lt	Papildymas, puslapis 78
es	Anexo, página 14	lv	Papildu dokumenti, lapp. 82
it	Supplemento, pagina 18	ro	Fișă de completare, pagina 86
ptBR	Documento suplementar, página 22	ru	Приложение, стр. 90
nl	Aanvullingsdocument, pagina 26	bg	Допълнение, Страница 94
da	Supplement, side 30	zh	增补, 第 98 页
no	Tilleggsdokument, side 34	ko	부록, 페이지 102
sv	Tilläggsinformation, sida 38	ja	補足説明書, ページ 106
cs	Dodatek, strana 42		
hr	Dodatni dokument, stranica 46		
sr	Dodatak, strana 50		
sk	Dodatok, strana 54		
sl	Priloga, stran 58		
hu	Kiegészítő dokumentum, oldal 62		
pl	Suplement, strona 66		

WARNING

To properly use this medical device,
read and comply with the
instructions for use and this
supplement.

Anesthesia workstation Software 3.n

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

Device	Part number	Edition
Fabius Tiro	9038686	from 1. Edition
Fabius GS <i>premium</i>	9038923	from 1. Edition
Fabius MRI	9039036	from 1. Edition
Fabius plus	9039054	from 1. Edition
Fabius <i>plus</i> XL	9052882	from 1. Edition
Fabius GS	9038685	from 1. Edition
Fabius CE	9037441	from 1. Edition

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System overview

Symbols

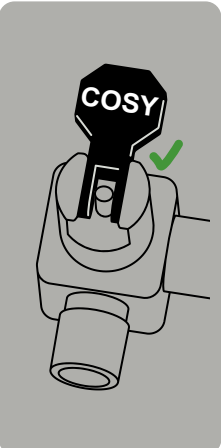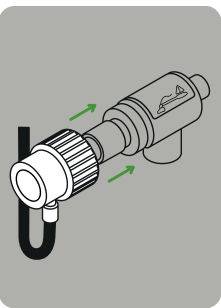	Use of the external fresh-gas outlet with an additional switch: When an automatic ventilation mode (Volume Control, Pressure Control, Pressure Support, SIMV/PS) or the ManSpont ventilation mode is used, set the switch on the external fresh-gas outlet to the COSY position. For further information, see "Using the external fresh-gas outlet with an additional switch (optional)" in the "Operation" chapter in the instructions for use.
	Using the external fresh-gas outlet as a common gas outlet: When an automatic ventilation mode (Volume Control, Pressure Control, Pressure Support, SIMV/PS) or the ManSpont ventilation mode is used, connect the short fresh-gas hose to the external fresh-gas outlet. For further information, see "Using the external fresh-gas outlet as a common gas outlet (optional)" in the "Operation" chapter in the instructions for use.

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Alarm – Cause – Remedy

Alarm – Cause – Remedy

Priority	Alarm message	Possible cause	Remedy
(31)	<i>INSP O₂ LOW !!!</i>	Inspiratory O ₂ concentration is below the lower alarm limit.	Check O ₂ supply. Check the setting of the flow control valve and lower O ₂ alarm limit.
		Incorrect setting of the switch on the external fresh-gas outlet or fresh-gas hose not connected to the external fresh-gas outlet.	Check the position of the switch or the connection of the fresh-gas hose to the external fresh-gas outlet.
(21)	<i>FRESH GAS LOW !!</i>	Insufficient fresh-gas supply in all ventilation modes.	Ensure sufficient fresh-gas supply.
		Hose blocked/kinked.	Check hoses.
		Leakage or disconnection in the breathing system.	Check breathing system.
		Incorrect setting of the switch on the external fresh-gas outlet or fresh-gas hose not connected to the external fresh-gas outlet.	Check the position of the switch or the connection of the fresh-gas hose to the external fresh-gas outlet.

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Fabius family

Supplement to the instructions for use

Device	Part number	Edition
Fabius Tiro	9054601	from 1. Edition
Fabius GS <i>premium</i>	9054626	from 1. Edition
Fabius MRI	9054649	from 1. Edition
Fabius plus	9054673	from 1. Edition
Fabius <i>plus</i> XL	9054695	from 1. Edition



- Keep this supplement with the instructions for use of the medical device.

The supplement updates the information of the instructions for use in the following chapters:

Overview

Product labels

The following product labels have been added.

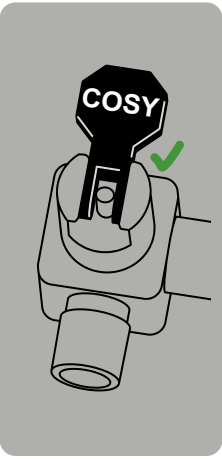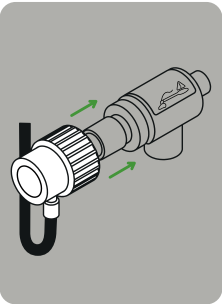
Product label	Explanation
	Use of the external fresh-gas outlet with an additional switch: When an automatic ventilation mode (Volume Control, Pressure Control, Pressure Support, SIMV/PS) or the ManSpont ventilation mode is used, set the switch on the external fresh-gas outlet to the COSY position. For further information, see "Using the external fresh-gas outlet with an additional switch (optional)" in the "Operation" chapter in the instructions for use.
	Using the external fresh-gas outlet as a common gas outlet: When an automatic ventilation mode (Volume Control, Pressure Control, Pressure Support, SIMV/PS) or the ManSpont ventilation mode is used, connect the short fresh-gas hose to the external fresh-gas outlet. For further information, see "Using the external fresh-gas outlet as a common gas outlet (optional)" in the "Operation" chapter in the instructions for use.

Operation

Using the external fresh-gas outlet as a common gas outlet (optional)

The following chapters have been changed:

Operation with the compact breathing system (COSY)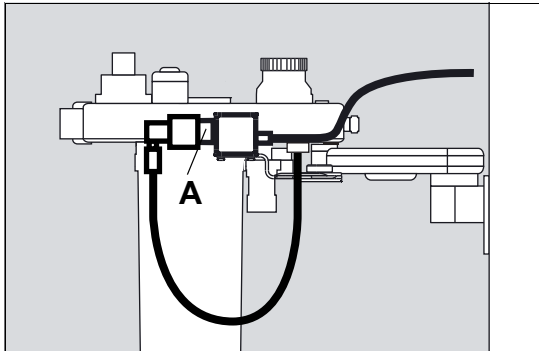

Before using an automatic ventilation mode (**Volume Control**, **Pressure Control**, **Pressure Support**, **SIMV/PS**) or the **ManSpont** ventilation mode:

- 1 Disconnect the non-rebreathing system from the external fresh-gas outlet.
- 2 Connect the fresh-gas hose to the external fresh-gas outlet (A).
- 3 Screw the sample line back to the Y-piece on the breathing circuit.


Ending operation

- 1 Close all flow control valves on the device.

Using the external fresh-gas outlet with an additional switch (optional)

The following chapters have been changed:

Operation with the compact breathing system (COSY)

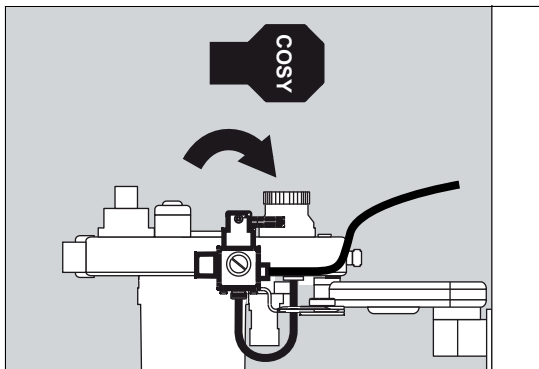- 1 Set the switch lever to **COSY**.

The lever points in the direction of the fresh-gas inlet.

- 2 Screw the sample line back to the Y-piece on the breathing circuit.

Ending operation

- 1 Close all flow control valves on the device.

Before using an automatic ventilation mode (**Volume Control**, **Pressure Control**, **Pressure Support**, **SIMV/PS**) or the **ManSpont** ventilation mode:

Troubleshooting

Fault – Cause – Remedy

The causes and remedies for the following alarms have been changed.

Alarm priority	Alarm	Cause	Remedy
(31)	INSP O₂ LOW !!!	Inspiratory O ₂ concentration is below the lower alarm limit.	Check O ₂ supply. Check the setting of the flow control valve and lower O ₂ alarm limit.
		Incorrect setting of the switch on the external fresh-gas outlet or fresh-gas hose not connected to the external fresh-gas outlet.	Check the position of the switch or the connection of the fresh-gas hose to the external fresh-gas outlet.
(21)	FRESH GAS LOW !!	Insufficient fresh-gas supply in all ventilation modes.	Ensure sufficient fresh-gas supply.
		Hose blocked/kinked.	Check hoses.
		Leakage or disconnection in the breathing system.	Check breathing system.
		Incorrect setting of the switch on the external fresh-gas outlet or fresh-gas hose not connected to the external fresh-gas outlet.	Check the position of the switch or the connection of the fresh-gas hose to the external fresh-gas outlet.

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© Drägerwerk AG & Co. KGaA