Hermann Bock GmbH – Nickelstr. 12 – 33415 Verl

To all National Competent Authorities of the Member States of the European Union (EU)

Verl, 19.11.2019

Field Safety Corrective Action ref. E-2019-0035-10426-Assmann

Dear Sirs and Madams,

We would like to inform you about a Field Safety Corrective Action (FSCA) for delivered beds with double drives in the period of January 2012 to Mai 2013. Certain double drives from the company limoss GmbH & Co.KG, type MD200, are affected. The circuit board of the described double drives is equipped with a diode, which disconnects the emergency lowering battery from the parallel connected power supply. In individual cases it could come to a defect of this diode, and therefore to a damage of the emergency lowering battery.

The corresponding Field Safety Corrective Action Report Form and the Field Safety Notice (FSN) can be found in the attachment.

The safety and satisfaction of our customers are our topmost priorities. Although only one claim has been registered, the company Hermann Bock is carrying out the present corrective measure. Clients who are using the corresponding drives, depending on the application, should carry out one of the described safety measures.

Our customers have already been informed so that the measures are currently being implemented. For further information, please do not hesitate to contact us.

Kind regards,

Hermann Bock GmbH

Klaus Bock
General Manager and Safety Officer for Medical Devices

Attachments:

Field Safety Corrective Action Report Form
Field Safety Notice with annex

Hermann Bock GmbH
Nickelstr. 12 · D - 33415 Verl · Germany
Phone: +49 (0)5246/9205-0
Fax: +49 (0)5246/9205-25
E-Mail: info@bock.net · www.bock.net

Managing Directors:
Klaus Bock, Meinolf Köhn, Werner Prühs
Local Court (Amtsgericht) Gütersloh: HRB 2654
VAT identification number: DE126790681
Creditor identifier: DE98ZZZ00000062516

Bank details:
Kreissparkasse Wiedenbrück
IBAN: DE54 4785 3520 0004 0004 77 · BIC: WELADED1WDB
Volksbank Bielefeld-Gütersloh eG
IBAN: DE55 4786 0125 0222 4446 00 · BIC: VBG TDE3MXXX

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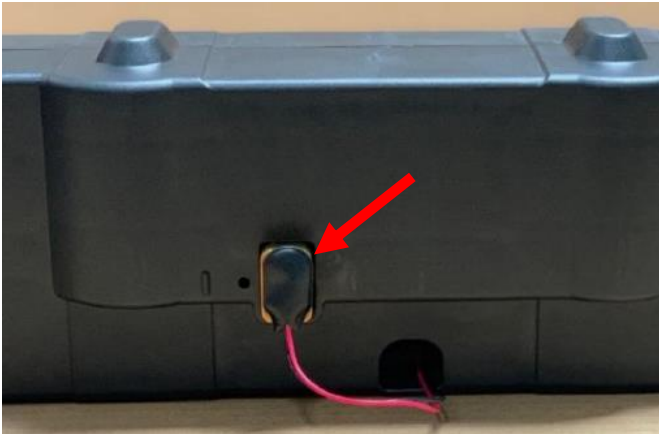
The following nursing care beds can be equipped with the above-described double drives:

- Children's nursing care bed kängbo
- all types of belluno incl. bed-in-bed-systems (bibs)
- all types of livorno
- all types of domiflex incl. adi.flex
- ancona
- eloflex
- valeo.flex
- centro.flex
- variolift
- verona
- Eco-bed

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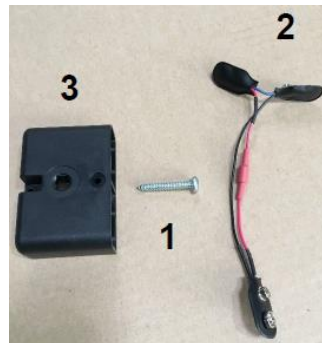
The retrofit can be done by the customer. Kindly check before the retrofit, if the article code and manufacturing date of your drives comply with the below mentioned indications of the effected drives! If one of the indications does not comply, do not carry out a retrofitting.

The emergency battery of the retrofit kit is secured with an additional diode in the connecting cable of the battery. In order to enable the retrofit of the double drive MD200 you receive the following parts:

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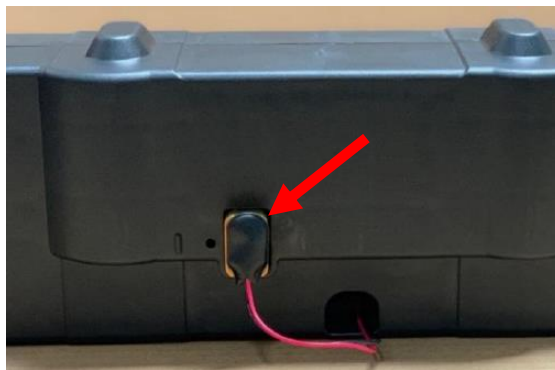
Kindly follow the below mentioned steps for the retrofit:

1. Removal of the old battery

To remove the old battery pull it out of the battery case of the drive and disconnect it from the connecting plugs.

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
2. Assembly of the battery holder

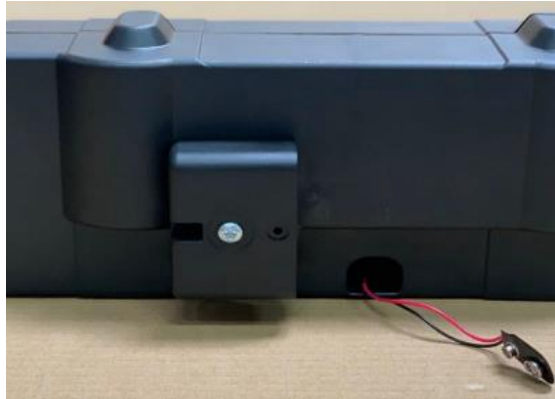To connect the new battery holder with the torx screw, please use the designated drilling next to the battery case. Fasten the screw with a hand screw driver. You would need a screw driver type „Torx T25“. Pay attention that you do not fasten the screw too tight. The casing of the holder should not deform.

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

3. Connection of the batteries

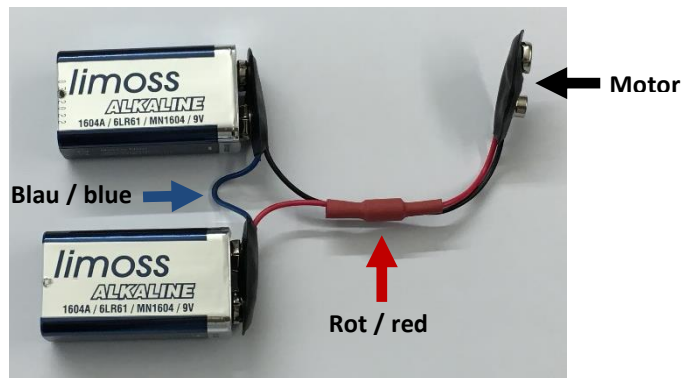Connect two 9 V monobloc batteries to the contacts of the provided retrofit cable. The contacts with the red and the black cable must be connected to the motor!

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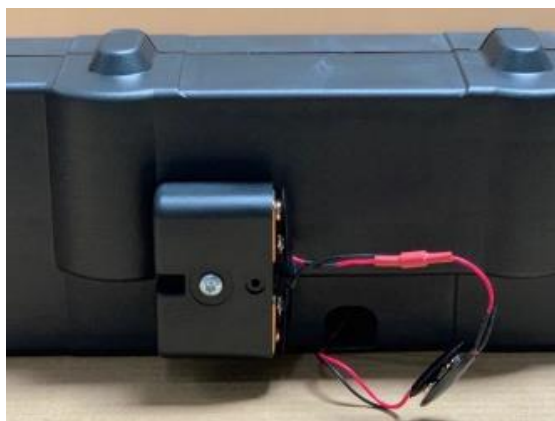Put the batteries into the fastened battery holder. Connect each cable plug (red and black cable) of the retrofit kit with the plugs coming out of the motor.

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The motor is now successfully retrofitted.

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Identification of the manufacturing date

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- This customer information is related to the double drive MD 200 with the article code 450792/800.00453
- The corrective measure concerns all drives that have been produced before the 5th calendar week 2013
- You recognize the manufacturing date as follows:
 Date: FCW03= manufacturing date of the drive
 E= 2012, F= 2013 CW = calendar week
 The corrective measure should be carried out for the drives with the identification marking:
 - ECW01 bis ECW52
 - FCW01 bis FCW04

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(limoss GmbH & Co. KG/ Her-