

To our customers of

Perseus A500 Ceiling Version mounted to Dräger Ceiling Supply Units

December 2021

Important Safety Notice

Potential drop of Perseus A500 from Dräger Ceiling Supply Units if system hits an obstacle and is manipulated after collision

Only the following products are affected:

Perseus A500 Ceiling Version in cases when used in combination with Dräger Ceiling Supply Units

Dear Sir or Madam,

During the course of our global market surveillance activities, we have become aware of a case in which a Perseus A500 Anesthesia workstation ceiling mounted version dropped from a Dräger Ceiling Supply Unit Movita Lift Strong.

The scenario of the event has been the following:

- User lowered down the Perseus A500 that was installed on the Movita Lift Strong.
- The Perseus A500 hit an obstacle which was located underneath.
- The Movita Lift Strong automatically stopped as intended due to the internal built in emergency stop
- After the emergency stop of the system, a mechanical manipulation was done at the ceiling supply unit. This caused an unintentional release of the Perseus A500 from the ceiling supply unit.

No patient nor user injury was reported but in case of recurrence it cannot be excluded that the anesthesia workstation may hit persons in the vicinity of the device.

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
Postal address:
23542 Lübeck, Germany
Tel +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
VAT no. DE135082211

Bank details:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift-Code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift-Code: NOLADE21SPL

Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7903 HL
General partner: Drägerwerk Verwaltungs AG
Registered office: Lübeck
Commercial register:
District Court of Lübeck HRB 7395 HL

Chairman of the Executive Board of
Drägerwerk AG & Co. KGaA and
Drägerwerk Verwaltungs AG:
Stefan Lauer
Executive Board:
Stefan Dräger (chairman)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske
Anton Schrófner


During further investigation after the event it has been confirmed that in all cases, when an object was hit by a Perseus A500 during lowering the Dräger Ceiling Supply Unit, the built-in emergency stop of the ceiling supply unit functioned properly and interrupted the movement. In case of emergency stop of a Movita ceiling supply unit a red indication is given at the Perseus A500 as well as at the ceiling supply unit.

In connection with a Forta ceiling supply unit there is no red indication in case of emergency stop.

Picture 1 Red Indication LED on the Perseus A500:



Picture 2 Movita Lift / Lift Strong

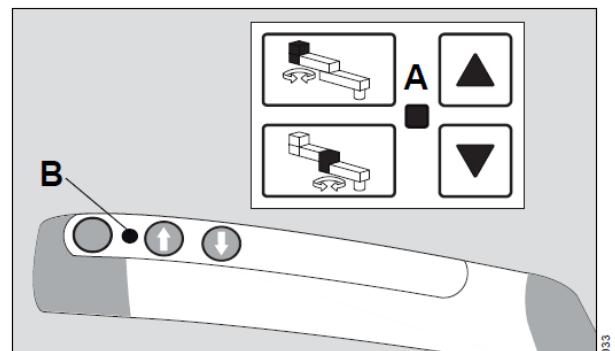

An **LED** indicator in the middle of the control panel (A) or on the control handles (B) shows the status of the system:

Nevertheless, we recognized that the specific sequence of action that needs to be followed by the user when a Perseus A500 is stuck on an object underneath the workstation while lowering the ceiling supply unit, has not been described sufficiently.

Actions to be taken:

During height adjustments,

1. Please make sure that no objects are placed under the Anesthesia Workstation
2. Check status of the above-mentioned indicators (Movita).

In case a Perseus A500 collided with an object underneath the anesthesia device:

3. Do not manipulate the ceiling supply unit nor the Perseus A500 by e.g. connecting /disconnecting devices; do not remove the obstacles.
4. Move up the ceiling supply unit with the connected anesthesia device, by using one of the switches at the ceiling supply unit or at the Perseus A500.
5. Remove any obstacle after the ceiling supply unit has been lifted and the collision has been resolved.

If these recommendations are followed the devices including the height adjustment feature can safely be used.

Please ensure that all users of the above-mentioned products and other persons within your organization are made aware of this Important Safety Notice and act accordingly. If you have provided the products to third parties, please forward a copy of this information.

Dräger is currently investigating how the connection interface can be made more robust against the above-mentioned course of handling. Within that scope we will update your Perseus A500 Anesthesia workstation ceiling mounted version free of charge with an improved magnet inside the emergency stop system. In addition, we will provide you with an Instruction for Use (IfU) amendment for the Dräger Ceiling Supply Units.

Please keep this information at least until the measures have been completed.

The responsible Competent authorities have been notified of this action.

Identification of the affected medical devices:

According to our records, you have received a Perseus A500 ceiling version manufactured by Drägerwerk AG & Co. KGaA (European Single Registration Number: DE-MF-000005329).

You are affected by this Safety notice if you use the Perseus A500 ceiling version in combination with Movita Lift / Lift Strong (G16707), Forta (G16810) or DVE 8080.

Only these Dräger combinations use the above-mentioned emergency stop system. In case your Perseus A500 is installed on the third-party ceiling unit, please contact the supplier of the ceiling supply unit for advice.

Contact:

If you have any questions, please do not hesitate to contact your local Dräger representative.

We apologize for any inconvenience caused by this measure, but we believe this to be a necessary measure to increase patient and your staff safety.


We thank you for your support.

With kind regards



Hans Ulrich Schüler

Product Management
Business Area Anesthesiology
Business Unit Therapy

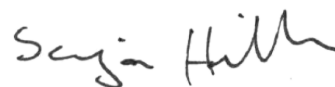

Sonja Hillmer

Post Market Surveillance
Quality & Regulatory Affairs
Medical Division

Attachment:

List of potentially affected medical devices

Australia	Brunei Dar-es-S	China	Croatia	Finland
ASHM-0115	ASHM-0030	ASMC-0181	ASNF-0097	ASNE-0039
Austria	ASHM-0037	ASMC-0182	ASNF-0111	ASNE-0040
ASHH-0015	ASKE-0007	ASMF-0018	ASNF-0112	ASNE-0129
ASHH-0016	ASKE-0009	ASMF-0019	ASNF-0113	ASNE-0130
ASHH-0017	Chile	ASMF-0020	ASNJ-0083	ASNE-0131
ASHH-0018	ASNH-0349	ASMF-0021	ASNJ-0084	ASNE-0132
ASKL-0017	ASNH-0350	ASMH-0248	ASNJ-0085	ASNE-0133
ASKL-0018	ASNH-0351	ASMH-0249	ASNJ-0086	ASNE-0134
ASKL-0026	ASNH-0352	ASMH-0250	Czech Republic	ASNE-0135
ASLJ-0084	ASNH-0353	ASMH-0251	ASJC-0069	ASPC-0040
ASLJ-0085	ASNH-0354	ASMH-0252	ASKM-0135	France
Belarus	ASNH-0355	ASMK-0010	ASKM-0136	ASEL-0096
ASKF-0048	ASNH-0356	ASMK-0011	ASPA-0174	ASEL-0097
ASKF-0049	China	ASMK-0012	ASPA-0175	ASEL-0105
ASKF-0050	ASFJ-0026	ASMK-0013	ASPA-0176	ASEL-0106
ASKF-0051	ASJH-0016	ASMK-0014	ASPA-0177	ASEL-0107
ASKF-0052	ASJL-0112	ASMK-0015	ASPA-0178	ASEM-0088
ASKL-0121	ASJN-0028	ASMK-0031	ASPB-0041	ASEM-0089
ASNC-0145	ASJN-0029	ASMK-0032	ASPB-0042	ASEM-0090
Belgium	ASJN-0114	ASMK-0033	ASPB-0043	ASEM-0094
ASEE-0049	ASJN-0115	ASMK-0034	ASPB-0044	ASEM-0095
ASEE-0050	ASJN-0116	ASMK-0035	ASPB-0045	ASEM-0096
ASEE-0051	ASJN-0117	ASMK-0036	Finland	ASEM-0110
ASEE-0052	ASJN-0118	ASMM-0030	ASHL-0041	ASEM-0111
ASEJ-0002	ASKD-0056	ASMM-0031	ASHL-0042	ASEM-0112
ASEJ-0003	ASKE-0079	ASMM-0045	ASHM-0038	ASEM-0120
ASEJ-0016	ASKE-0080	ASMM-0046	ASNA-0109	ASEN-0034
ASFJ-0067	ASKF-0013	ASMM-0047	ASNA-0110	ASEN-0072
ASFH-0053	ASKM-0141	ASMM-0048	ASND-0121	ASEN-0072
ASFH-0055	ASKN-0020	ASNA-0183	ASND-0122	ASFE-0005
ASFH-0056	ASKN-0024	ASNB-0215	ASND-0123	ASFF-0033
ASFH-0057	ASKN-0060	ASNH-0187	ASND-0124	ASFN-0002
ASFJ-0038	ASLK-0022	ASNH-0188	ASND-0125	ASHA-0039
ASFJ-0039	ASLK-0056	ASNH-0189	ASND-0126	ASHA-0040
ASFJ-0040	ASLL-0188	ASNH-0190	ASND-0127	ASHA-0041
ASFJ-0041	ASLN-0043	ASNH-0191	ASND-0189	ASHA-0042
ASFJ-0042	ASLN-0044	ASNH-0196	ASND-0190	ASHA-0043
ASFJ-0043	ASLN-0045	ASNH-0197	ASND-0198	ASHA-0044
ASFJ-0044	ASLN-0046	Columbia	ASND-0217	ASHF-0065
ASFJ-0048	ASLN-0047	ASHN-0069	ASNE-0016	ASHJ-0012
ASHJ-0054	ASLN-0053	ASLE-0047	ASNE-0017	ASHL-0066
ASHJ-0055	ASMA-0027	ASLE-0048	ASNE-0018	ASHI-0068

ASJB-0130	ASMA-0028	Croatia	ASNE-0033	ASJH-0069
ASJB-0131	ASMA-0029	ASMD-0226	ASNE-0034	ASJK-0072
ASJB-0132	ASMA-0069	ASNF-0093	ASNE-0035	ASJK-0073
ASLJ-0018	ASMA-0070	ASNF-0094	ASNE-0036	ASJK-0076
ASLJ-0019	ASMA-0071	ASNF-0095	ASNE-0037	ASJK-0077
ASNH-0363	ASMB-0037	ASNF-0096	ASNE-0038	ASJK-0078
France	Germany	Germany	Germany	Germany
ASJK-0079	ASEN-0061	ASJN-0060	ASMC-0224	ASML-0042
ASJK-0080	ASEN-0066	ASKB-0048	ASMD-0001	ASML-0043
ASJK-0081	ASEN-0067	ASKB-0053	ASMD-0023	ASML-0044
ASJK-0096	ASEN-0068	ASKB-0055	ASMD-0024	ASML-0045
ASJK-0111	ASEN-0069	ASKC-0095	ASME-0018	ASML-0046
ASJK-0112	ASEN-0070	ASKC-0096	ASME-0019	ASML-0047
ASJK-0113	ASEN-0071	ASKC-0097	ASMK-0098	ASML-0057
ASJK-0114	ASEN-0073	ASKC-0103	ASMK-0099	ASML-0058
ASJK-0119	ASFD-0012	ASKJ-0134	ASMK-0100	ASML-0059
ASJM-0022	ASFD-0027	ASKM-0156	ASMK-0108	ASML-0068
ASJM-0075	ASFK-0041	ASKN-0001	ASMK-0109	ASML-0069
ASJM-0076	ASFM-0146	ASKN-0016	ASMK-0110	ASML-0070
ASJM-0077	ASFM-0147	ASKN-0018	ASMK-0111	ASML-0071
ASJM-0078	ASHB-0084	ASJN-0045	ASMK-0116	ASML-0086
ASJM-0168	ASHF-0122	ASJN-0049	ASMK-0117	ASML-0087
ASKA-0044	ASHH-0056	ASJN-0060	ASMK-0118	ASML-0088
ASKD-0007	ASHJ-0031	ASKB-0048	ASMK-0166	ASML-0098
ASKH-0053	ASHJ-0032	ASKB-0053	ASMK-0167	ASML-0099
ASKH-0105	ASHL-0043	ASKB-0055	ASMK-0168	ASML-0109
ASKH-0106	ASHL-0045	ASKC-0095	ASMK-0173	ASML-0110
ASKH-0108	ASHL-0046	ASKC-0096	ASMK-0174	ASML-0120
ASKK-0024	ASHL-0047	ASKC-0097	ASMK-0175	ASML-0121
ASLF-0138	ASHL-0048	ASKC-0103	ASMK-0177	ASML-0122
ASLM-0182	ASJC-0066	ASKJ-0134	ASMK-0178	ASML-0123
ASLM-0183	ASJC-0067	ASKM-0156	ASMK-0179	ASML-0124
ASLN-0155	ASJC-0071	ASKN-0001	ASMK-0192	ASML-0125
ASMD-0251	ASJE-0055	ASKN-0016	ASMK-0193	ASML-0144
ASMD-0252	ASJE-0119	ASKN-0018	ASMK-0194	ASML-0145
ASMD-0262	ASJJ-0104	ASKN-0019	ASMK-0201	ASML-0146
ASNC-0064	ASJJ-0105	ASLE-0081	ASMK-0202	ASML-0155
ASNC-0065	ASJJ-0106	ASLE-0090	ASMK-0203	ASML-0156
ASNC-0066	ASJJ-0107	ASLM-0190	ASMK-0204	ASML-0171
ASNC-0067	ASJJ-0108	ASLM-0191	ASMK-0205	ASML-0172
ASNH-0111	ASJJ-0109	ASLM-0192	ASMK-0206	ASML-0173
ASPE-0004	ASJJ-0110	ASLM-0193	ASMK-0219	ASML-0174
Germany	ASJJ-0111	ASLN-0164	ASMK-0220	ASML-0175
ASEE-0053	ASJJ-0112	ASLN-0165	ASMK-0221	ASML-0177
ASEF-0007	ASJJ-0113	ASLN-0166	ASML-0006	ASML-0188
ASEF-0008	ASJJ-0114	ASLN-0167	ASML-0007	ASML-0189
ASEF-0009	ASJJ-0115	ASMA-0001	ASML-0008	ASML-0190
ASEF-0010	ASJK-0100	ASMA-0002	ASML-0018	ASML-0203
ASEF-0011	ASJM-0094	ASMA-0003	ASML-0019	ASML-0204

ASEF-0018	ASJM-0095	ASMA-0004	ASML-0020	ASML-0205
ASEF-0020	ASJM-0096	ASMA-0005	ASML-0021	ASMM-0146
ASEF-0021	ASJM-0097	ASMA-0013	ASML-0022	ASNA-0179
ASEF-0022	ASJM-0098	ASMA-0015	ASML-0023	ASNA-0180
ASEF-0023	ASJM-0108	ASMA-0026	ASML-0033	ASNA-0181
ASEJ-0012	ASJN-0043	ASMB-0192	ASML-0034	ASNC-0138
ASEL-0055	ASJN-0044	ASMB-0217	ASML-0035	ASNE-0217
ASEL-0056	ASJN-0045	ASMB-0225	ASML-0036	ASNE-0218
ASEN-0055	ASJN-0049	ASMC-0223	ASML-0041	ASNE-0219
Germany	Ireland	Italy	Italy	
ASNF-0038	ASJC-0013	ASHN-0013	ASLL-0113	
ASNF-0039	Italy	ASHN-0014	ASLL-0117	
ASNF-0040	ASFA-0036	ASHN-0027	ASLL-0118	
ASNJ-0124	ASFA-0037	ASJB-0049	ASLL-0119	
ASNK-0047	ASFA-0038	ASJB-0127	ASLM-0019	
ASNK-0066	ASFB-0011	ASJB-0139	ASLN-0093	
ASNK-0067	ASFB-0012	ASJB-0140	ASLN-0094	
ASNK-0068	ASFB-0013	ASJB-0141	ASMD-0209	
ASNK-0069	ASFB-0014	ASJB-0142	ASMD-0269	
ASNK-0070	ASFF-0051	ASJB-0144	ASMD-0270	
ASNK-0120	ASFF-0055	ASJB-0145	ASMH-0012	
ASNL-0052	ASFJ-0001	ASJB-0146	ASMJ-0003	
ASNL-0053	ASFJ-0020	ASJB-0147	ASMJ-0004	
ASNL-0054	ASFM-0036	ASJB-0149	ASMJ-0005	
ASNL-0055	ASFM-0037	ASJB-0150	ASMJ-0006	
ASNM-0028	ASHA-0045	ASJB-0151	ASMJ-0007	
ASNM-0029	ASHA-0046	ASJC-0012	ASMJ-0097	
ASNM-0030	ASHA-0118	ASJC-0030	ASMJ-0098	
ASPA-0076	ASHA-0120	ASJC-0031	ASMM-0037	
ASPE-0026	ASHA-0121	ASJE-0079	ASMM-0038	
ASPF-0138	ASHA-0124	ASJE-0080	ASMM-0039	
Greece	ASHA-0125	ASJF-0018	ASMM-0040	
ASFF-0012	ASHA-0126	ASJF-0019	ASNB-0217	
ASFF-0013	ASHA-0127	ASJF-0020	ASNB-0218	
ASFF-0014	ASHA-0128	ASJF-0021	ASNB-0219	
ASFF-0015	ASHB-0033	ASJF-0022	ASNC-0143	
ASFF-0016	ASHB-0035	ASJF-0023	ASNC-0208	
India	ASHB-0036	ASJM-0079	ASPB-0021	
ASHB-0004	ASHB-0037	ASJM-0163	ASPB-0022	
ASHB-0005	ASHB-0038	ASJM-0164	ASPD-0063	
ASKM-0150	ASHB-0039	ASJM-0167	ASPE-0050	
ASKM-0151	ASHB-0040	ASJN-0068	ASPE-0051	
ASKM-0152	ASHB-0043	ASJN-0071	ASPE-0052	
ASLC-0166	ASHC-0076	ASJN-0072	ASPE-0053	
ASLC-0167	ASHC-0078	ASJN-0080	ASPE-0054	
ASLC-0168	ASHC-0080	ASJN-0081	ASPJ-0004	
ASLC-0169	ASHC-0081	ASJN-0082	Japan	
ASLJ-0051	ASHC-0082	ASJN-0083	ASFH-0085	
ASLJ-0052	ASHC-0083	ASKE-0027	ASKH-0040	

ASMD-0264
Indonesia
 ASFH-0050
 ASLD-0122
 ASLD-0123
 ASLD-0124
 ASLD-0125
 ASLD-0126
Iran
 ASFM-0171
 ASFM-0172
 ASFN-0018
 ASJB-0072

ASHC-0084
 ASHC-0085
 ASHC-0086
 ASHC-0087
 ASHC-0088
 ASHH-0050
 ASHH-0053
 ASHM-0003
 ASHM-0004
 ASHM-0066
 ASHM-0010
 ASHN-0011
 ASHN-0012

ASKE-0033
 ASKE-0037
 ASKE-0044
 ASKE-0121
 ASKE-0122
 ASKK-0034
 ASKL-0016
 ASLC-0138
 ASLF-0057
 ASLH-0089
 ASLH-0090
 ASLL-0050
 ASLL-0051

ASKN-0056
 ASNJ-0186
 ASNJ-0187
 ASPB-0003
 ASPC-0084
 ASPC-0085
Kuwait
 ASFF-0030
 ASFF-0031
 ASFF-0032
 ASJM-0045
 ASJM-0046
 ASJM-0047

Kuwait
 ASKB-0098
 ASKB-0099
 ASKB-0100
 ASKB-0102
 ASKB-0104
 ASKB-0106
 ASKB-0107
 ASKB-0109
 ASKC-0006
 ASKC-0007
 ASKC-0013
 ASKC-0016
 ASKD-0099
 ASLM-0079
 ASLM-0080
 ASLM-0081
 ASLM-0082
 ASLM-0090
 ASLM-0091
 ASLM-0092
 ASLM-0093
 ASLM-0094
Lithuania
 ASFM-0128
 ASFM-0129
 ASJN-0001
Mexico
 ASKD-0015
 ASKE-0082
 ASKH-0078
 ASKH-0079
 ASLL-0065
 ASLL-0066
 ASMK-0001
 ASMM-0238

Netherlands
 ASHD-0023
 ASHD-0024
 ASHD-0027
 ASHD-0028
 ASHD-0029
 ASHL-0056
 ASJJ-0097
 ASJJ-0118
 ASJJ-0119
 ASJK-0035
 ASJK-0036
 ASJK-0037
 ASJK-0038
 ASJK-0039
 ASJK-0040
 ASJK-0041
 ASJK-0042
 ASJK-0043
 ASJK-0044
 ASKA-0010
 ASKA-0011
 ASKA-0013
 ASKA-0014
 ASKA-0016
 ASKA-0017
 ASKA-0018
 ASLD-0115
 ASLD-0116
 ASMC-0194
 ASMC-0195
 ASMC-0221
 ASMC-0222
 ASNH-0299
 ASNH-0300
 ASNH-0301

Norway
 ASHE-0020
 ASHF-0016
 ASHF-0017
 ASJE-0081
 ASJE-0082
 ASJE-0083
 ASKC-0021
 ASKC-0022
 ASKC-0025
 ASKC-0026
 ASLA-0052
 ASLA-0053
 ASLA-0058
 ASLA-0070
 ASLA-0072
 ASLB-0104
 ASLB-0105
 ASLF-0059
 ASMJ-0067
 ASMJ-0068
 ASMJ-0069
 ASMJ-0070
 ASMJ-0071
 ASMJ-0072
 ASMJ-0082
 ASMJ-0083
 ASMJ-0099
 ASMJ-0100
 ASMK-0059
 ASMK-0060
 ASMK-0061
 ASMK-0062
 ASMK-0063
 ASMK-0070
 ASMK-0071

Norway
 ASNA-0008
 ASNA-0009
 ASNA-0010
 ASNA-0011
 ASNA-0013
 ASNA-0014
 ASNA-0015
 ASNA-0024
 ASNA-0025
 ASNA-0026
 ASNA-0027
 ASNA-0028
 ASNA-0029
 ASNA-0030
 ASNA-0031
 ASNA-0034
 ASNA-0035
 ASNA-0036
 ASNA-0037
 ASNA-0178
 ASNC-0152
 ASNC-0153
 ASNC-0161
 ASNC-0162
 ASNC-0163
 ASNC-0164
 ASNC-0165
 ASNC-0166
 ASNE-0125
 ASNE-0126
 ASNE-0127
 ASNE-0128
 ASNE-0136
 ASNE-0137
 ASNE-0138

ASMM-0239
 ASMM-0240
 ASMM-0241
 ASNH-0017
 ASNH-0018
 ASNH-0019
 ASNH-0020
 ASNH-0021
 ASPE-0105
 ASPE-0106
Morocco
 ASMH-0113
 ASMH-0114
 ASMH-0115
 ASMH-0116
 ASMH-0117
 ASNK-0006
 ASNK-0007

Panama

ASKK-0062
 ASKK-0071
 ASKK-0074
 ASKK-0076
 ASMF-0103

Peru

ASFL-0045

Poland

ASFL-0107
 ASFL-0108
 ASFL-0113
 ASFL-0114
 ASFL-0115
 ASLK-0008
 ASLK-0009
 ASLK-0010
 ASLK-0011
 ASLK-0013
 ASLK-0014
 ASLM-0124
 ASLM-0125
 ASLM-0126
 ASLM-0140
 ASLM-0145
 ASMK-0078
 ASMK-0079
 ASMK-0080
 ASMK-0081
 ASMK-0082
 ASNH-0297

ASNH-0302

Norway

ASFK-0033
 ASFL-0008
 ASFL-0009
 ASFL-0010
 ASFL-0011
 ASFL-0012
 ASFL-0013
 ASHD-0030
 ASHD-0031
 ASHE-0010
 ASHE-0012
 ASHE-0014
 ASHE-0015
 ASHE-0016
 ASHE-0018
 ASHE-0019

Portugal

ASKK-0150
 ASKK-0154
 ASKK-0155
 ASKK-0158
 ASKL-0002
 ASKL-0003
 ASKL-0004
 ASKL-0008
 ASKL-0009
 ASKL-0012
 ASKL-0013
 ASKL-0015
 ASKL-0025
 ASKL-0027
 ASKL-0031
 ASKL-0043
 ASKL-0073
 ASKL-0075
 ASKL-0080
 ASKL-0081
 ASKL-0082
 ASKL-0083
 ASLJ-0160
 ASME-0044
 ASME-0045
 ASME-0046
 ASME-0047
 ASME-0048
 ASME-0049
 ASME-0050

ASMK-0072
 ASMK-0073
 ASMK-0074
 ASMM-0099
 ASMM-0100
 ASMM-0101
 ASMM-0102
 ASMM-0103
 ASMM-0187
 ASMM-0207
 ASMN-0131
 ASMN-0132
 ASNA-0002
 ASNA-0003
 ASNA-0004
 ASNA-0005
 ASNA-0006
 ASNA-0007

Portugal

ASNA-0046
 ASNH-0284
 ASNH-0285
 ASNH-0286
 ASPA-0205
 ASPA-0206

Romania

ASHL-0125
 ASHL-0126

ASMM-0061

ASMM-0062

Russian Fed.

ASJK-0053
 ASNH-0095
 ASNH-0096
 ASPC-0064

Saudi Arabia

ASEJ-0023
 ASJH-0019
 ASJH-0020
 ASJH-0021
 ASJH-0022
 ASJH-0023
 ASJH-0024
 ASJH-0025
 ASJH-0026
 ASKJ-0132
 ASKJ-0135
 ASKJ-0141

ASNK-0075
 ASNK-0076
 ASNK-0077
 ASNM-0007
 ASPC-0104
 ASPC-0105
 ASPD-0096
 ASPD-0097
 ASPD-0099
 ASPD-0100
 ASPD-0101
 ASPD-0102
 ASPD-0104
 ASPD-0105
 ASPD-0106
 ASPD-0107
 ASPD-0108
 ASPE-0010
 ASPE-0021

ASNH-0373

Portugal

ASHL-0122

ASHL-0123

ASHL-0124

ASHM-0064

ASKF-0120

ASKF-0121

ASKF-0125

ASKF-0128

ASKK-0122

ASKK-0123

ASKK-0128

ASKK-0129

ASKK-0132

ASKK-0133

ASKK-0135

ASKK-0136

ASKK-0146

ASKK-0147

ASKK-0148

ASKK-0149

ASME-0051

ASME-0052

ASMF-0044

ASMF-0045

ASMF-0046

ASMF-0047

ASMF-0048

ASMF-0049

ASMF-0050

ASMF-0051

ASMK-0022

ASMK-0023

ASMK-0024

ASMK-0025

ASNA-0038

ASNA-0039

ASNA-0040

ASNA-0041

ASNA-0042

ASNA-0043

ASNA-0044

ASNA-0045