

2025-09-11

URGENT FIELD SAFETY NOTICE

Manufacturer SRN: DE-MF-000020091

FSCA Reference: 1365554 HU 35 – Potential mycobacterial contamination

FSN Type: New

Affected Product: Heater Unit HU 35, 230V (Mat. 701072162)
Heater Unit HU 35, 110-115V (Mat. 701072163)

Unique Device Identifier: 04058863041193 (Mat. 701072162)
04058863041308 (Mat. 701072163)

Affected Serial No.: Refer to Annex I

For Attention of: Users of the medical device listed above

Dear valued customer,

Maquet Cardiopulmonary GmbH (MCP) would like to inform users of the Heater Unit HU 35 about a potential contamination with non-tuberculosis mycobacteria (NTM).

The HU 35 acts as a heat supply in order to maintain the patient's body temperature via a heat exchanger of a PLS oxygenator or HLS Module or other oxygenator-heat exchangers as part of extracorporeal circuits.

Problem Description

HU 35 units are disinfected at the end of production; however, they are not produced in a sterile environment. It is important to note that these units are not classified as sterile medical devices. Both the Instruction for Use (IFU) and the Service Manual state this information.

Due to the recent relocation of the production site in June 2024, a proactive decision was made to manufacture additional HU 35 units in advance to mitigate potential supply disruptions. As a result, the actual storage duration for the affected units extended to 4 up to 12 months.

The extended storage period increases the risk of bacterial growth. While disinfection at the end of production reduces microbial load, it may not eliminate all bacteria in the circuitry. Bacteria present in the devices may proliferate over the prolonged storage time, contributing to contamination.

In course of a manufacturer rework, initiated on 21 July 2025, HU 35 units from stock were disinfected and sampled prior to distribution. On 12 August 2025, MCP became aware of a unit which was tested positive for mycobacteria after disinfection. Later, further units were tested positive for mycobacteria. Further, MCP has received four complaints regarding contaminations of the water circuit of HU 35 devices that were part of the pre-produced stock with *Mycobacteria chimaera*.

Therefore, the scope of this field action is limited to all devices delivered after 1 June 2024.

Hazardous situation

A Health Hazard Evaluation is currently in progress. Based on current assessments:

The HU 35 water circuit is a closed system, meaning the water does not come into direct contact with the patient. Potential transmission may occur indirectly, for example, through insufficient hygiene practices by clinical staff. The oxygenator circuit is also a closed system. Its integrity is verified prior to use through a priming process. The device has been confirmed not to emit aerosols, effectively excluding airborne bacterial transmission.

Given these findings, MCP has provisionally identified the following hazardous situations that may arise:

- Exposure to pathogenic agents (patient, user, third person)

Potential harm

The possible immediate and/or long-range health consequences and risk levels of the nonconformance include the following:

- Infection

As mentioned, Maquet Cardiopulmonary GmbH has identified four customer complaints, however, there was no patient harm, serious injuries, or deaths reported.

In this context, it is worth to mention that since market introduction of the HU 35, MCP has not been notified of any infection in a patient caused by bacterial contamination of the device worldwide.

Corrective Action to be taken by the user:**For devices currently used in treatment:**

- Upon termination of treatment, the customer shall take the HU 35 devices affected out of operation at the earliest opportunity and perform:
 - The descaling process of the water circuit with 2% citric acid according to chapter 8.2.1 of the IFU (including subsequent 3 rinses of the device)
 - The high-level disinfection process of the water circuit with a concentration of 5% Chloramine T according to chapter 8.2.3 of the IFU.
- Further, customers are recommended to perform testing of the units affected for NTM after performing this high-level disinfection, prior to any further patient use.
- Please note that by using common PCR analysis also dead NTM or DNA fragments of them lead to positive results. This does not necessarily indicate an inadequate disinfection. Therefore, MCP recommends performing cultural detection tests for the presence of NTM.
- If there are still atypical mycobacteria in the water system detectable, please repeat the high-level disinfection.
- Afterwards, follow the usual disinfection protocol as per IFU.

For devices currently not used in treatment/ not installed:

- The customer shall immediately perform:
 - The descaling process of the water circuit with 2% citric acid according to chapter 8.2.1 of the IFU (including subsequent 3 rinses of the device)
 - The high-level disinfection process of the water circuit with a concentration of 5% Chloramine T according to chapter 8.2.3 of the IFU.
- Further, customers are recommended to perform testing of the units affected for NTM after performing this high-level disinfection, prior to any further patient use.
- Please note that by using common PCR analysis also dead NTM or DNA fragments of them lead to positive results. This does not necessarily indicate an inadequate disinfection. Therefore, MCP recommends performing cultural detection tests for the presence of NTM.
- If there are still atypical mycobacteria in the water system detectable, please repeat the high-level disinfection.
- Afterwards, follow the usual disinfection protocol as per IFU.

Action to be taken by user:

- ☒ Identify Device
☐ Return Device

- ☐ Quarantine Device
☐ Destroy Device

Details on further action(s):

- According to our post-market surveillance documentation, you may have products affected by this action. Please examine your inventory **immediately** to determine, if you have any affected product in your inventory.
- Regardless of whether you have the affected product(s) or not, duly fill out the enclosed Customer Response Form and return it to your local Getinge representative **as soon as possible, but no later than 10 October 2025**, by mentioning **FSCA-1365554** as reference in the subject line of your mail.
- Upon receipt of the signed Customer Response Form, Getinge will provide Credit Note for the necessary descaling and disinfection agent.
- Please **always** report any adverse events, e.g., infections potentially related to the affected products, to your Getinge representative.
- Patients should be monitored as appropriate according to your facility's standard of care.

Actions to be taken by the manufacturer:

- ☐ Product Removal
☐ Software Upgrade
☒ Other

- ☐ On-site device modification/ inspection
☐ IFU or labeling change
☐ None

- Inform all customers possessing the affected products **promptly** about this Field Action by sending the Field Safety Notice for Customers.
- Upon receipt of the signed Customer Response Form, provide Credit Note for the necessary descaling and disinfection agent.
- Finalize root cause investigation to determine whether further action is needed.

Enclosed documents:

- Customer Response Form
- Annex I List of affected products

Transmission of the Field Safety Notice:

- Please ensure in your organization that all users of the above-mentioned products and other persons to be informed are made aware of this Urgent Field Safety Notice.
- Please transfer this notice to other organizations on which the action has an impact.
- If you have given the products to third parties, please forward a copy of this information or inform the contact person indicated below.
- Please maintain awareness on the notice and resulting actions for an appropriate period to ensure effectiveness of the corrective action.

We sincerely apologize for any inconvenience this may cause you and we will do our utmost to carry through this action as swiftly as possible.

As required, we have provided this notification to the necessary Regulatory Agencies.

Should you have questions or require additional information, please contact your local Getinge representative.

Sincerely,

Vice President

Signature: *Dieter Engel*

Electronically signed by: Dieter Engel
Reason: I approve this document.
Date: Sep 11, 2025 09:06:28 EDT

Email: dieter.engel@getinge.com

Person Responsible for Regulatory Compliance (PRRC)

Signature: *Alexander Bernhardt*

Electronically signed by: Alexander Bernhardt
Reason: I approve this document.
Date: Sep 11, 2025 15:00:39 GMT+2

Email: alexander.bernhardt@getinge.com

Contact details of manufacturer

Maquet Cardiopulmonary GmbH
Kehler Str. 31
76437 Rastatt
GERMANY
Phone: +49 7222 932 - 0
Email: FSCA.cp@getinge.com

CUSTOMER RESPONSE FORM

FSCA Reference: 1365554 HU 35 – Potential mycobacterial contamination

Affected Product: Heater Unit HU 35, 230V (Mat. 701072162)
Heater Unit HU 35, 110-115V (Mat. 701072163)

Affected Serial No.: Refer to Annex I

Please send this form at the latest by **10 October 2025**, to your local Getinge representative.

By completing this document and signing it, I acknowledge that I have read and understand the following associated points:

- I have read and understand this Field Safety Notice. We will take action as soon as possible according to given instructions.
 - I confirm that I have distributed this Field Safety Notice to the affected personal.
- ☐ I do not have any affected products in my inventory:
- ☐ I have the following affected products in my inventory:

Article Number	Description	Serial Number	Quantity
XXXXXX.XXXX	<SAP Product name>		

Your Comments:

Country

Hospital / Clinic (full address)

Date

Name (Function)

Signature

Please return the completed form to your local Getinge representative by email [enter local Getinge mail address or via post enter local Getinge address or FAX>](#):

Annex I List of affected products

This Annex I List of affected products is considered as a supplementary attachment to the 1365554 Field Safety Notice.

Below are listed all lots of products which are affected and have been distributed.

Country	Article Number	Description	Serial Number	Quantity
AE	701072162	Heater Unit HU 35, 230V	90034641	1
AE	701072162	Heater Unit HU 35, 230V	90034893	1
AE	701072162	Heater Unit HU 35, 230V	90034995	1
AE	701072162	Heater Unit HU 35, 230V	90035015	1
AE	701072162	Heater Unit HU 35, 230V	90035082	1
AL	701072162	Heater Unit HU 35, 230V	90034890	1
AM	701072162	Heater Unit HU 35, 230V	90034950	1
AT	701072162	Heater Unit HU 35, 230V	90034925	1
AT	701072162	Heater Unit HU 35, 230V	90034929	1
AT	701072162	Heater Unit HU 35, 230V	90034933	1
AT	701072162	Heater Unit HU 35, 230V	90034999	1
AU	701072162	Heater Unit HU 35, 230V	90034596	1
AU	701072162	Heater Unit HU 35, 230V	90034597	1
AU	701072162	Heater Unit HU 35, 230V	90034598	1
AU	701072162	Heater Unit HU 35, 230V	90034642	1
AU	701072162	Heater Unit HU 35, 230V	90034644	1
AU	701072162	Heater Unit HU 35, 230V	90034672	1
AU	701072162	Heater Unit HU 35, 230V	90034690	1
AU	701072162	Heater Unit HU 35, 230V	90034692	1
AU	701072162	Heater Unit HU 35, 230V	90034694	1
AU	701072162	Heater Unit HU 35, 230V	90034703	1
AU	701072162	Heater Unit HU 35, 230V	90034705	1
AU	701072162	Heater Unit HU 35, 230V	90034706	1

Country	Article Number	Description	Serial Number	Quantity
AU	701072162	Heater Unit HU 35, 230V	90034710	1
AU	701072162	Heater Unit HU 35, 230V	90034714	1
AU	701072162	Heater Unit HU 35, 230V	90034715	1
AU	701072162	Heater Unit HU 35, 230V	90034723	1
AU	701072162	Heater Unit HU 35, 230V	90034875	1
AU	701072162	Heater Unit HU 35, 230V	90034889	1
AU	701072162	Heater Unit HU 35, 230V	90034909	1
BH	701072162	Heater Unit HU 35, 230V	90034828	1
CA	701072163	Heater Unit HU 35, 110-115V	90034748	1
CA	701072163	Heater Unit HU 35, 110-115V	90034749	1
CA	701072163	Heater Unit HU 35, 110-115V	90034750	1
CA	701072163	Heater Unit HU 35, 110-115V	90034751	1
CA	701072163	Heater Unit HU 35, 110-115V	90034752	1
CA	701072163	Heater Unit HU 35, 110-115V	90034753	1
CA	701072163	Heater Unit HU 35, 110-115V	90034755	1
CA	701072163	Heater Unit HU 35, 110-115V	90034758	1
CA	701072163	Heater Unit HU 35, 110-115V	90034762	1
CA	701072163	Heater Unit HU 35, 110-115V	90034763	1
CA	701072163	Heater Unit HU 35, 110-115V	90034765	1
CA	701072163	Heater Unit HU 35, 110-115V	90034766	1
CA	701072163	Heater Unit HU 35, 110-115V	90034767	1
CA	701072163	Heater Unit HU 35, 110-115V	90034768	1
CA	701072163	Heater Unit HU 35, 110-115V	90034783	1
CA	701072163	Heater Unit HU 35, 110-115V	90034785	1
CH	701072162	Heater Unit HU 35, 230V	90034662	1
CH	701072162	Heater Unit HU 35, 230V	90034667	1
CH	701072162	Heater Unit HU 35, 230V	90034695	1

Country	Article Number	Description	Serial Number	Quantity
CH	701072162	Heater Unit HU 35, 230V	90034696	1
CH	701072162	Heater Unit HU 35, 230V	90034699	1
CH	701072162	Heater Unit HU 35, 230V	90034704	1
CN	701072162	Heater Unit HU 35, 230V	90034634	1
CN	701072162	Heater Unit HU 35, 230V	90034638	1
CN	701072162	Heater Unit HU 35, 230V	90034639	1
CN	701072162	Heater Unit HU 35, 230V	90034640	1
CN	701072162	Heater Unit HU 35, 230V	90034653	1
CN	701072162	Heater Unit HU 35, 230V	90034654	1
CN	701072162	Heater Unit HU 35, 230V	90034658	1
CN	701072162	Heater Unit HU 35, 230V	90034660	1
CN	701072162	Heater Unit HU 35, 230V	90034663	1
CN	701072162	Heater Unit HU 35, 230V	90034664	1
CN	701072162	Heater Unit HU 35, 230V	90034666	1
CN	701072162	Heater Unit HU 35, 230V	90034670	1
CN	701072162	Heater Unit HU 35, 230V	90034671	1
CN	701072162	Heater Unit HU 35, 230V	90034675	1
CO	701072162	Heater Unit HU 35, 230V	90034712	1
CR	701072163	Heater Unit HU 35, 110-115V	90034760	1
CR	701072163	Heater Unit HU 35, 110-115V	90034761	1
CZ	701072162	Heater Unit HU 35, 230V	90034701	1
CZ	701072162	Heater Unit HU 35, 230V	90034713	1
CZ	701072162	Heater Unit HU 35, 230V	90034720	1
CZ	701072162	Heater Unit HU 35, 230V	90034631	1
CZ	701072162	Heater Unit HU 35, 230V	90034636	1
CZ	701072162	Heater Unit HU 35, 230V	90034637	1
CZ	701072162	Heater Unit HU 35, 230V	90034830	1

Country	Article Number	Description	Serial Number	Quantity
CZ	701072162	Heater Unit HU 35, 230V	90034846	1
CZ	701072162	Heater Unit HU 35, 230V	90035093	1
DE	701072162	Heater Unit HU 35, 230V	90034786	1
DE	701072162	Heater Unit HU 35, 230V	90034633	1
DE	701072162	Heater Unit HU 35, 230V	90034635	1
DE	701072162	Heater Unit HU 35, 230V	90034643	1
DE	701072162	Heater Unit HU 35, 230V	90034646	1
DE	701072162	Heater Unit HU 35, 230V	90034674	1
DE	701072162	Heater Unit HU 35, 230V	90034677	1
DE	701072162	Heater Unit HU 35, 230V	90034683	1
DE	701072162	Heater Unit HU 35, 230V	90034685	1
DE	701072162	Heater Unit HU 35, 230V	90034687	1
DE	701072162	Heater Unit HU 35, 230V	90034691	1
DE	701072162	Heater Unit HU 35, 230V	90034693	1
DE	701072162	Heater Unit HU 35, 230V	90034717	1
DE	701072162	Heater Unit HU 35, 230V	90034787	1
DE	701072162	Heater Unit HU 35, 230V	90034788	1
DE	701072162	Heater Unit HU 35, 230V	90034790	1
DE	701072162	Heater Unit HU 35, 230V	90034794	1
DE	701072162	Heater Unit HU 35, 230V	90034795	1
DE	701072162	Heater Unit HU 35, 230V	90034796	1
DE	701072162	Heater Unit HU 35, 230V	90034797	1
DE	701072162	Heater Unit HU 35, 230V	90034798	1
DE	701072162	Heater Unit HU 35, 230V	90034799	1
DE	701072162	Heater Unit HU 35, 230V	90034802	1
DE	701072162	Heater Unit HU 35, 230V	90034803	1
DE	701072162	Heater Unit HU 35, 230V	90034805	1

Country	Article Number	Description	Serial Number	Quantity
DE	701072162	Heater Unit HU 35, 230V	90034806	1
DE	701072162	Heater Unit HU 35, 230V	90034808	1
DE	701072162	Heater Unit HU 35, 230V	90034809	1
DE	701072162	Heater Unit HU 35, 110-115V	90034810	1
DE	701072162	Heater Unit HU 35, 110-115V	90034811	1
DE	701072162	Heater Unit HU 35, 110-115V	90034812	1
DE	701072162	Heater Unit HU 35, 110-115V	90034813	1
DE	701072162	Heater Unit HU 35, 110-115V	90034814	1
DE	701072162	Heater Unit HU 35, 110-115V	90034815	1
DE	701072162	Heater Unit HU 35, 110-115V	90034816	1
DE	701072162	Heater Unit HU 35, 110-115V	90034821	1
DE	701072162	Heater Unit HU 35, 110-115V	90034825	1
DE	701072162	Heater Unit HU 35, 110-115V	90034829	1
DE	701072162	Heater Unit HU 35, 110-115V	90034831	1
DE	701072162	Heater Unit HU 35, 110-115V	90034832	1
DE	701072162	Heater Unit HU 35, 110-115V	90034833	1
DE	701072162	Heater Unit HU 35, 110-115V	90034834	1
DE	701072162	Heater Unit HU 35, 110-115V	90034835	1
DE	701072162	Heater Unit HU 35, 110-115V	90034841	1
DE	701072162	Heater Unit HU 35, 230V	90034842	1
DE	701072162	Heater Unit HU 35, 230V	90034844	1
DE	701072162	Heater Unit HU 35, 230V	90034845	1
DE	701072162	Heater Unit HU 35, 230V	90034847	1
DE	701072162	Heater Unit HU 35, 230V	90034877	1
DE	701072162	Heater Unit HU 35, 230V	90034882	1
DE	701072162	Heater Unit HU 35, 230V	90034885	1
DE	701072162	Heater Unit HU 35, 230V	90034886	1

Country	Article Number	Description	Serial Number	Quantity
DE	701072162	Heater Unit HU 35, 230V	90034888	1
DE	701072162	Heater Unit HU 35, 230V	90034892	1
DE	701072162	Heater Unit HU 35, 230V	90034894	1
DE	701072162	Heater Unit HU 35, 230V	90034896	1
DE	701072162	Heater Unit HU 35, 230V	90034899	1
DE	701072162	Heater Unit HU 35, 230V	90034908	1
DE	701072162	Heater Unit HU 35, 230V	90034910	1
DE	701072162	Heater Unit HU 35, 230V	90034915	1
DE	701072162	Heater Unit HU 35, 230V	90034916	1
DE	701072162	Heater Unit HU 35, 230V	90034918	1
DE	701072162	Heater Unit HU 35, 230V	90034920	1
DE	701072162	Heater Unit HU 35, 230V	90034921	1
DE	701072162	Heater Unit HU 35, 230V	90034926	1
DE	701072162	Heater Unit HU 35, 110-115V	90034936	1
DE	701072162	Heater Unit HU 35, 110-115V	90034937	1
DE	701072162	Heater Unit HU 35, 230V	90034938	1
DE	701072162	Heater Unit HU 35, 230V	90034939	1
DE	701072162	Heater Unit HU 35, 230V	90034940	1
DE	701072162	Heater Unit HU 35, 230V	90034941	1
DE	701072162	Heater Unit HU 35, 230V	90034942	1
DE	701072162	Heater Unit HU 35, 230V	90034944	1
DE	701072162	Heater Unit HU 35, 230V	90034948	1
DE	701072162	Heater Unit HU 35, 230V	90035089	1
DZ	701072162	Heater Unit HU 35, 230V	90034959	1
DZ	701072162	Heater Unit HU 35, 230V	90034963	1
ES	701072162	Heater Unit HU 35, 230V	90034895	1
ES	701072162	Heater Unit HU 35, 230V	90034903	1

Country	Article Number	Description	Serial Number	Quantity
ES	701072162	Heater Unit HU 35, 230V	90034907	1
ES	701072162	Heater Unit HU 35, 230V	90034949	1
ES	701072162	Heater Unit HU 35, 230V	90034954	1
ES	701072162	Heater Unit HU 35, 230V	90034966	1
FI	701072162	Heater Unit HU 35, 230V	90034665	1
FI	701072162	Heater Unit HU 35, 230V	90034676	1
FI	701072162	Heater Unit HU 35, 230V	90034686	1
FI	701072162	Heater Unit HU 35, 230V	90035091	1
FR	701072162	Heater Unit HU 35, 230V	90034657	1
FR	701072162	Heater Unit HU 35, 230V	90034700	1
FR	701072162	Heater Unit HU 35, 230V	90034708	1
FR	701072162	Heater Unit HU 35, 230V	90034718	1
FR	701072162	Heater Unit HU 35, 230V	90034719	1
FR	701072162	Heater Unit HU 35, 230V	90034721	1
FR	701072162	Heater Unit HU 35, 230V	90034817	1
FR	701072162	Heater Unit HU 35, 230V	90034818	1
FR	701072162	Heater Unit HU 35, 230V	90034820	1
FR	701072162	Heater Unit HU 35, 230V	90034822	1
FR	701072162	Heater Unit HU 35, 230V	90034826	1
FR	701072162	Heater Unit HU 35, 230V	90034827	1
FR	701072162	Heater Unit HU 35, 230V	90034836	1
FR	701072162	Heater Unit HU 35, 230V	90034837	1
FR	701072162	Heater Unit HU 35, 230V	90034838	1
FR	701072162	Heater Unit HU 35, 230V	90034848	1
FR	701072162	Heater Unit HU 35, 230V	90034854	1
FR	701072162	Heater Unit HU 35, 110-115V	90034856	1
FR	701072162	Heater Unit HU 35, 110-115V	90034859	1

Country	Article Number	Description	Serial Number	Quantity
FR	701072162	Heater Unit HU 35, 110-115V	90034862	1
FR	701072162	Heater Unit HU 35, 110-115V	90034863	1
FR	701072162	Heater Unit HU 35, 110-115V	90034865	1
FR	701072162	Heater Unit HU 35, 110-115V	90034866	1
FR	701072162	Heater Unit HU 35, 110-115V	90034867	1
FR	701072162	Heater Unit HU 35, 110-115V	90034869	1
FR	701072162	Heater Unit HU 35, 110-115V	90034871	1
FR	701072162	Heater Unit HU 35, 110-115V	90034874	1
FR	701072162	Heater Unit HU 35, 110-115V	90034881	1
FR	701072162	Heater Unit HU 35, 110-115V	90034883	1
FR	701072162	Heater Unit HU 35, 110-115V	90034884	1
FR	701072162	Heater Unit HU 35, 110-115V	90034913	1
FR	701072162	Heater Unit HU 35, 110-115V	90034930	1
FR	701072162	Heater Unit HU 35, 110-115V	90034945	1
FR	701072162	Heater Unit HU 35, 230V	90034951	1
FR	701072162	Heater Unit HU 35, 230V	90034984	1
FR	701072162	Heater Unit HU 35, 230V	90035037	1
GB	701072162	Heater Unit HU 35, 230V	90034887	1
GB	701072162	Heater Unit HU 35, 230V	90034891	1
GB	701072162	Heater Unit HU 35, 230V	90034900	1
GB	701072162	Heater Unit HU 35, 230V	90034912	1
HR	701072162	Heater Unit HU 35, 230V	90034722	1
HR	701072162	Heater Unit HU 35, 230V	90034876	1
HR	701072162	Heater Unit HU 35, 230V	90034880	1
ID	701072162	Heater Unit HU 35, 230V	90034901	1
IL	701072162	Heater Unit HU 35, 230V	90034648	1
IL	701072162	Heater Unit HU 35, 230V	90034655	1

Country	Article Number	Description	Serial Number	Quantity
IL	701072162	Heater Unit HU 35, 230V	90034804	1
IT	701072162	Heater Unit HU 35, 230V	90034645	1
IT	701072162	Heater Unit HU 35, 230V	90034688	1
IT	701072162	Heater Unit HU 35, 230V	90034931	1
IT	701072162	Heater Unit HU 35, 230V	90034934	1
IT	701072162	Heater Unit HU 35, 230V	90034935	1
IT	701072162	Heater Unit HU 35, 230V	90034947	1
IT	701072162	Heater Unit HU 35, 230V	90034958	1
KR	701072162	Heater Unit HU 35, 110-115V	90034587	1
KR	701072162	Heater Unit HU 35, 110-115V	90034588	1
KR	701072162	Heater Unit HU 35, 230V	90034589	1
KR	701072162	Heater Unit HU 35, 230V	90034590	1
KR	701072162	Heater Unit HU 35, 230V	90034591	1
KR	701072162	Heater Unit HU 35, 230V	90034592	1
KR	701072162	Heater Unit HU 35, 230V	90034599	1
KR	701072162	Heater Unit HU 35, 230V	90034668	1
KR	701072162	Heater Unit HU 35, 230V	90034669	1
KR	701072162	Heater Unit HU 35, 230V	90034673	1
KR	701072162	Heater Unit HU 35, 230V	90034689	1
KR	701072162	Heater Unit HU 35, 230V	90034697	1
KR	701072162	Heater Unit HU 35, 230V	90034707	1
KR	701072162	Heater Unit HU 35, 230V	90034711	1
KR	701072162	Heater Unit HU 35, 230V	90034824	1
KR	701072162	Heater Unit HU 35, 230V	90034850	1
KR	701072162	Heater Unit HU 35, 230V	90034851	1
KR	701072162	Heater Unit HU 35, 230V	90034852	1
KR	701072162	Heater Unit HU 35, 230V	90034853	1

Country	Article Number	Description	Serial Number	Quantity
KR	701072162	Heater Unit HU 35, 230V	90034857	1
KR	701072162	Heater Unit HU 35, 230V	90034858	1
KR	701072162	Heater Unit HU 35, 230V	90034860	1
KR	701072162	Heater Unit HU 35, 230V	90034861	1
KR	701072162	Heater Unit HU 35, 230V	90034872	1
KR	701072162	Heater Unit HU 35, 230V	90034873	1
KR	701072162	Heater Unit HU 35, 230V	90034878	1
KR	701072162	Heater Unit HU 35, 230V	90034879	1
KR	701072162	Heater Unit HU 35, 230V	90034902	1
KR	701072162	Heater Unit HU 35, 230V	90034946	1
KR	701072162	Heater Unit HU 35, 230V	90034956	1
KR	701072162	Heater Unit HU 35, 230V	90034957	1
KR	701072162	Heater Unit HU 35, 230V	90034967	1
KR	701072162	Heater Unit HU 35, 230V	90034968	1
KR	701072162	Heater Unit HU 35, 230V	90034994	1
KR	701072162	Heater Unit HU 35, 230V	90035001	1
KR	701072162	Heater Unit HU 35, 230V	90035004	1
KW	701072162	Heater Unit HU 35, 230V	90034855	1
KW	701072162	Heater Unit HU 35, 230V	90034923	1
KW	701072162	Heater Unit HU 35, 230V	90034927	1
KW	701072162	Heater Unit HU 35, 110-115V	90034964	1
KW	701072162	Heater Unit HU 35, 110-115V	90034965	1
LY	701072162	Heater Unit HU 35, 110-115V	90034839	1
LY	701072162	Heater Unit HU 35, 110-115V	90034969	1
MA	701072162	Heater Unit HU 35, 110-115V	90034911	1
MA	701072162	Heater Unit HU 35, 110-115V	90034955	1
MA	701072162	Heater Unit HU 35, 110-115V	90034962	1

Country	Article Number	Description	Serial Number	Quantity
MA	701072162	Heater Unit HU 35, 110-115V	90034983	1
MX	701072163	Heater Unit HU 35, 110-115V	90034786	1
MX	701072163	Heater Unit HU 35, 110-115V	90034779	1
MX	701072163	Heater Unit HU 35, 110-115V	90034780	1
MX	701072163	Heater Unit HU 35, 110-115V	90034782	1
MX	701072163	Heater Unit HU 35, 110-115V	90034784	1
MY	701072162	Heater Unit HU 35, 110-115V	90034975	1
NI	701072163	Heater Unit HU 35, 110-115V	90034757	1
NZ	701072162	Heater Unit HU 35, 110-115V	90034698	1
NZ	701072162	Heater Unit HU 35, 230V	90034716	1
OM	701072162	Heater Unit HU 35, 230V	90035002	1
PL	701072162	Heater Unit HU 35, 230V	90034840	1
PL	701072162	Heater Unit HU 35, 230V	90034843	1
PL	701072162	Heater Unit HU 35, 230V	90034981	1
PL	701072162	Heater Unit HU 35, 230V	90034996	1
PL	701072162	Heater Unit HU 35, 230V	90035000	1
PT	701072162	Heater Unit HU 35, 230V	90034943	1
RO	701072162	Heater Unit HU 35, 230V	90034924	1
SA	701072162	Heater Unit HU 35, 230V	90034647	1
SA	701072162	Heater Unit HU 35, 230V	90034650	1
SA	701072162	Heater Unit HU 35, 230V	90034651	1
SA	701072162	Heater Unit HU 35, 230V	90034652	1
SA	701072162	Heater Unit HU 35, 230V	90034656	1
SA	701072162	Heater Unit HU 35, 230V	90034659	1
SA	701072162	Heater Unit HU 35, 230V	90034849	1
SA	701072162	Heater Unit HU 35, 230V	90034864	1
SA	701072162	Heater Unit HU 35, 230V	90034898	1

Country	Article Number	Description	Serial Number	Quantity
SA	701072162	Heater Unit HU 35, 230V	90034904	1
SA	701072162	Heater Unit HU 35, 230V	90034905	1
SA	701072162	Heater Unit HU 35, 230V	90034906	1
SA	701072162	Heater Unit HU 35, 110-115V	90034919	1
SA	701072162	Heater Unit HU 35, 110-115V	90034960	1
SA	701072162	Heater Unit HU 35, 230V	90035011	1
SA	701072162	Heater Unit HU 35, 230V	90035032	1
SA	701072162	Heater Unit HU 35, 230V	90035033	1
SE	701072162	Heater Unit HU 35, 230V	90034702	1
SE	701072162	Heater Unit HU 35, 230V	90034979	1
SE	701072162	Heater Unit HU 35, 230V	90034985	1
SE	701072162	Heater Unit HU 35, 230V	90034986	1
SE	701072162	Heater Unit HU 35, 230V	90034987	1
SE	701072162	Heater Unit HU 35, 230V	90034988	1
SV	701072163	Heater Unit HU 35, 230V	90034777	1
TH	701072162	Heater Unit HU 35, 230V	90035079	1
TH	701072162	Heater Unit HU 35, 230V	90035107	1
UA	701072162	Heater Unit HU 35, 230V	90034819	1
VN	701072162	Heater Unit HU 35, 230V	90034649	1
VN	701072162	Heater Unit HU 35, 230V	90034709	1
VN	701072162	Heater Unit HU 35, 230V	90034897	1
VN	701072162	Heater Unit HU 35, 230V	90034922	1
VN	701072162	Heater Unit HU 35, 230V	90035092	1
ZA	701072162	Heater Unit HU 35, 230V	90034789	1
ZA	701072162	Heater Unit HU 35, 230V	90034791	1
ZA	701072162	Heater Unit HU 35, 230V	90034792	1